



**Dental Focus™, LLC**

P.O. Box 454, Neshanic Station, NJ 08853

P: (908) 968-3553 • Email: [info@dentalfocus.biz](mailto:info@dentalfocus.biz) • Web: [www.dentalfocus.biz](http://www.dentalfocus.biz)

Dear Customer:

To ensure that your new **Dental Focus Digital Dental X-ray Sensor** performs as promised, please review this final **Dental Sensor Checklist**. Please sign and return this document to email [orders@dentalfocus.biz](mailto:orders@dentalfocus.biz).

- Veterinary digital radiography requires hands-on positioning training. Please contact **Dental Focus™, LLC** if you need additional training. We can recommend many options and levels of training.
- Please contact **Dental Focus™, LLC** if you need **Remote Software Training**. This requires a **computer** with *high-speed Internet service*. **To schedule remote training call (908) 968-3553.**
- Follow these tips to ensure *proper care* of the **Dental Focus Digital Dental X-ray Sensor**:
  - Never leave an animal unattended, for any reason with the sensor in their mouth. We have only a small percentage of bit sensors with the majority of bites resulting from multi-tasking. We recommend removing the sensor from the animal’s mouth as soon as you are done shooting the X-ray, and then marking the X-rays after sensor is removed. We need to consider the many X-rays we take over time and eliminate the chance for an unattended animal causing damage to the sensor. We recommend you assign one individual to be in charge and ensure that this rule is always followed. If you have new employees, we will be happy to re-train as needed. Most damaged sensors can be avoided with this simple step.
  - Our software trainer will show you ways to limit or eliminate trips to the computer while shooting images. It is not appropriate to boot a computer or spend any time away from the animal with the sensor in the mouth.
  - If a patient must be kept light while under anesthesia, use a plastic mouth gag when shooting dental X-rays. The rule of thumb should be that if you would not put your hand in the animal’s mouth, the sensor should not be put in the animal’s mouth.
  - Always use **Protective Sensor Sheaths** in order to protect the sensor from moisture. NOT using **Protective Sensor Sheaths** voids sensor warranty. Contact **Dental Focus™, LLC** when you need to purchase additional **Protective Sensor Sheaths**
  - For additional protection, we recommend **wrapping your sheath-covered sensor with Vet Wrap** to prevent the sensor plate from slipping in the patient’s mouth. Using **Vet Wrap** also helps prevent scratches and helps to keep your sensor looking like new.
  - Mount the sensor to avoid dropping.
  - Do not have sensor cord crossing active walkways.
  - Never kink cords or tug/pull the sensor cord to remove the sensor from the animal’s mouth.
  - Contact your insurance company to confirm if you are covered for damage (accidental) outside the manufacturer’s warranty.
  - For **Dental Focus Dental Sensor/Software Technical Support**, call **(908) 968-3553**.

Review all information on this **Dental Focus Dental Sensor Checklist** with all current/new employees who use the **Dental Focus Digital Dental X-ray Sensor**. Please provide your signature below acknowledging the **Dental Focus Dental Sensor Checklist** and email it to [orders@dentalfocus.biz](mailto:orders@dentalfocus.biz).

Signature: \_\_\_\_\_ Date: \_\_\_\_\_

Print Hospital Name & Doctor Name: \_\_\_\_\_

Print Hospital Address: \_\_\_\_\_